


For PCNL

Accordion CoAx™ Stone Control Device

Simplify
and Shorten
*Percutaneous
Nephrolithotomy*
Procedures

Finally, a device to provide either
retrograde or antegrade stone fragment control over
a guidewire during percutaneous nephrolithotomy...

It's About Time!®

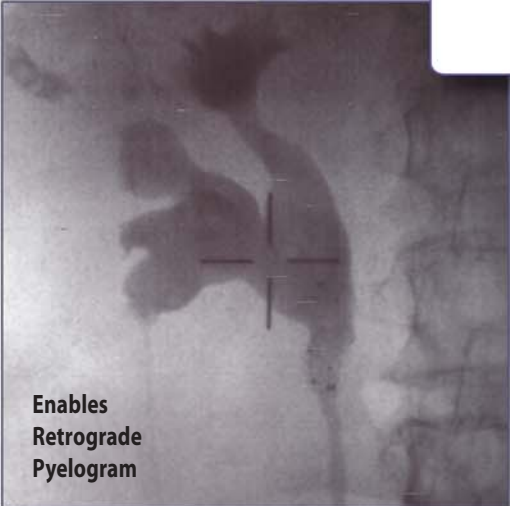

Enables
Retrograde
Pyelogram

PercSys®
INNOVATIONS IN UROLOGY

Accordion CoAx™

Stone Control Device

The 15mm Accordion CoAx Stone Control Device provides physicians a means to prevent antegrade stone fragment migration during percutaneous nephrolithotomy (PCNL).

- The Accordion CoAx device can be **introduced over a 0.035" or 0.038" guidewire** using either an antegrade or retrograde approach
- The large, compliant 15mm diameter occlusion **conforms to and fills most structures** in the proximal urinary tract and is designed to prevent migration of stone fragments into the ureter
- Unlike balloon catheters, the multi-fold film occlusion and over-the-wire design combine to **provide a stable platform** that is not easily displaced during the fragmentation process
- Radiopaque shaft and film markers allow easy **monitoring of device advancement and deployment** using fluoroscopy
- The short, tapered tip and hydrophilic coating facilitate **atraumatic entry** either through the skin or into the ureteral orifice
- The handle's ratcheting slider provides **positive tactile feedback** of the occlusion film deployment and undeployment




Antegrade Access

Using an antegrade approach, the low profile shaft allows introduction either alongside or inside the nephrostomy sheath

Retrograde Access

Using a retrograde approach, the guidewire can be removed and contrast media may be injected through the central lumen to conduct a retrograde pyelogram for diagnosis and targeting

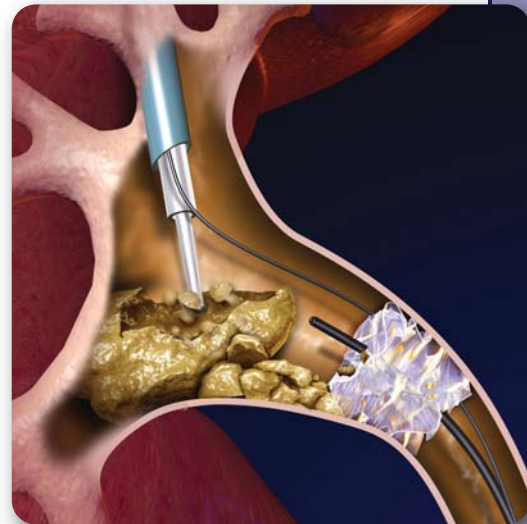

Ordering Information

Accordion CoAx Stone Control Device

Deployed OD: (nominal)	Length:	Guidewire Compatibility:	Shaft OD:	Order No:†
15mm	65cm	0.035" or 0.038"	4Fr	PA1305-01-15

† Available from Customer Service in boxes of 3 only

PercSys

3260 Hillview Avenue, Suite 100

Palo Alto, CA 94304

Toll Free: 1.877.987.6633

Tel: 1.650.493.4200

Fax: 1.650.493.4212

www.percsys.com

PercSys®
INNOVATIONS IN UROLOGY